


Confidence in Your Results

Please read all of the InSure® FIT™ Product Instructions and follow accordingly before performing the test procedure.

TEST PROCEDURE

IMPORTANT Do not touch Test Strip or Test Card with reagent bottles.


1 Remove the back panel of the Test Card.

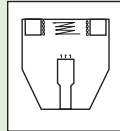


2 When handling the Test Strip, avoid the middle area of the Test Strip (nitrocellulose portion). Use the end *opposite of the arrows* at all times while handling the Test Strip.




3 Insert the Test Strip so that it securely locks into the cut-out on the Test Card (Note precautions on handling and storage of Test Strips).

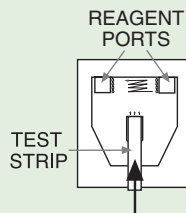

4 Add Run Buffer, one drop at a time alternating between reagent ports, until each port has received 4 drops (~150 µL).

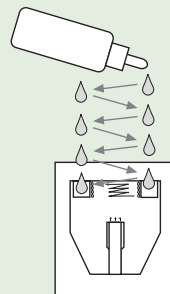


5 Wait 5 minutes and read the results.

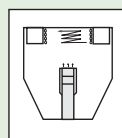


Note: If the Run Buffer does not flow onto the Test Strip due to excess specimen, add 2 more drops of buffer to each reagent port then wait an additional 5 minutes.

INTERPRETATION OF TEST RESULTS

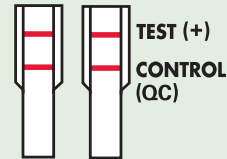


Positive Test (DETECTED)

Presence of fecal blood has been detected if both the Test and Control lines, including any trace in the Test Line area, are visible on the Test Strip.

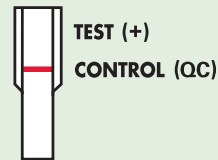

Negative Test (UNDETECTED)


No fecal blood has been detected if only the Control Line, and no trace of the Test Line, is visible.

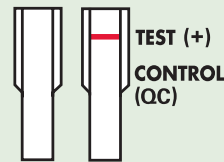

Invalid Test

Test is invalid and should be repeated with a new specimen Test Card if the Control Line does not appear. If the card does not show any reaction, add a few more drops of Run Buffer.

Actual Test Card with **positive results** shown

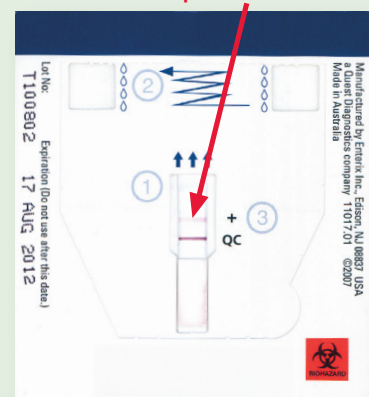

If you need technical assistance, please contact Technical Services, Enterix, Inc. at **1-800-531-3681** in the U.S.

InSure® FIT™
Fecal Immunochemical Test
Colorectal Cancer Screening
That's Easier to Live With